BACK UP™

Improve your posture and
increase your productivity

BQE
BACK QUALITY ERGONOMICS
STOOLS & CHAIRS

All BQE work stools and chairs are designed in such a way that it is easy to adopt a better sitting posture while working, which gives the stools/chairs a clear preventative and curative character. Development and production are based on the general scientific background, specific studies and practical experience.

The Back Up™

An ergonomic chair with an oblique seat part to sit more close to the patient or / and the microscope and with maximal back support.

Sitting in a healthy posture shows a trunk-knee angle (or angle between upper and lower legs) of 110° or more (hydraulic adjustable), so that a slight hollow curvature of the spine is obtained, comparable with a more standing position.

The backward rounding of the spine is avoided to prevent overloading.

To obtain a balanced sitting posture the seat bones rest on the horizontal part of the seat and the thighs on an oblique part.

The counter pressure (cp) of the oblique part is adjustable through the hydraulic button (a).

There is also a mechanic version and a fixed version. See internet www.bqe.nl

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EN If you've chosen the armrests, then the stool is already prepared for them.

BQE 01 / BQE CM

Place the armrest in the coupling under the stool, adjust the distance and fix it with knob 1. Determine the height of the armrests by pulling up and releasing knob 2.

BQE 02

Place the angle piece in the coupling under the stool, adjust the distance and fix it with knob 1. Place the armrest in the angle piece, turn the knob 2 loose, determine the height of the armrest and fix it with knob 2. The right angel position can be adjusted by repositioning bolt 3.

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EN Take the spare parts out of the box and place them side by side.

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EN Place the casters (1) in the base (2). Place the gas cylinder (3) and/or cylinder cover (5), with the thicker end down, in the base. If you have chosen a foot ring (4) and/or cylinder cover (5), place it over the gas cylinder.

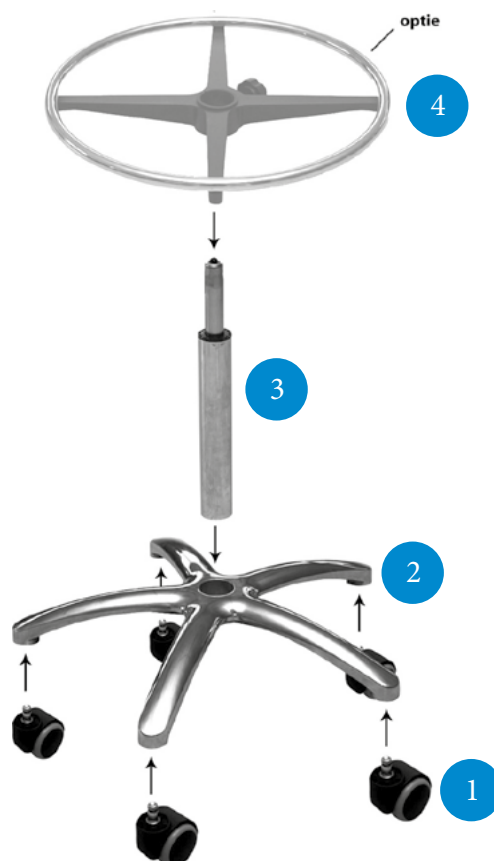
• If you have chosen the armrests, the stool has already been prepared for them. See page 5.

• If you have chosen the foot control, then it has already been assembled. All you have to do is to place the seat onto the gas cylinder.

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
EN Place the seat onto the base. The frame will fasten itself automatically once you sit on the seat. Turn the star knob completely loose in order to fit the back support. Then place the back support into the frame and tighten the star knob. You will use this knob later to adjust the height of the back support.

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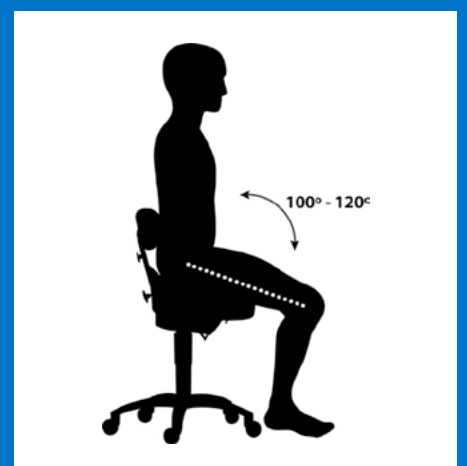EN First adjust the height of the stool with the handle (1). You should have an angle of 100° to 120° between the trunk and the thighs. Sit on the stool with the buttocks against the raised edge. Then adjust the height of the back support with the upper star knob (2). Now adjust the pressure of the support in your back with the lower star knob (3). Finally, adjust the resistance of the tilting mechanism with the knob (4). Loosen this knob a couple of turns, move the knob up or down and turn it tight again. Position 1 provides the lowest resistance and position 8 provides the highest resistance.

For more information about the seat height and adjustment see the instruction video at www.BQErgonomics.eu

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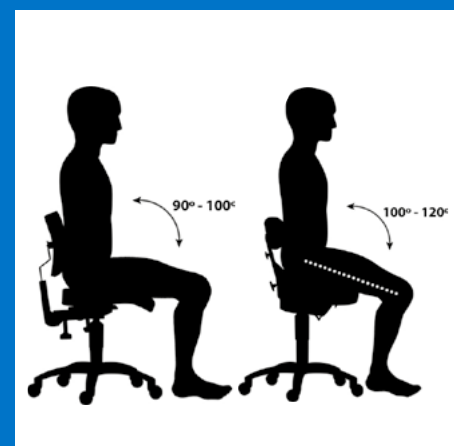
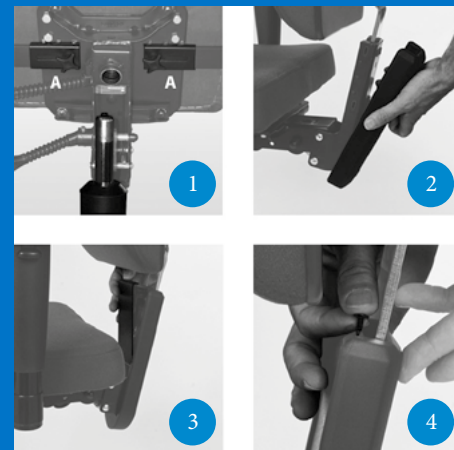
EN Place the seat onto the base (1). Place the back rest into the frame. The frame will fasten itself automatically once you sit on the seat. If you have chosen the armrests, put them into the grooves (A) under the frame. Place the protection cover on the back side of the back rest (2). Slide the front plate down the slot (3). Click both parts together. Push the two black pins on the upper side of the cover down for locking (4).

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EN First adjust the height of the stool with the handle (5). Sit on the stool with you back against the back rest. You should have an angle of 90° to 100° between the trunk and the thighs. Next adjust the seat depth by pushing up the handle (6). You can fix the right position by pushing down the handle. Now adjust the height of the back support by pulling it upwards, while doing it don't lean on the back support (7). To adjust the back support lower, first pull it completely to the top, and then slide it completely to the bottom. From this position it is possible to adjust the back support in steps. • The most commonly setting used is after 3 of 4 clicks.

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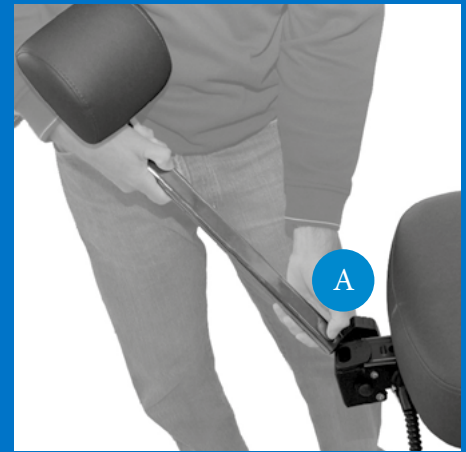
EN Place the seat onto the base. The frame will fasten itself automatically once you sit on the seat. Put the back support into the frame. For the 'De Luxe' models you must press the catch (A) at the bottom of the bar while adjusting. Then tighten the adjusting knob (3). You will use this knob later to adjust the height of the back support.

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
EN Adjust the height of the stool with the handle (1). You should have an angle of 100° to 120° between the trunk and the thighs. Adjust the seat depth and the pressure of the support in your back with the handle (2). Adjust the height of the back support with the knob (3). Adjust the slope angle of the seat with the lever (4). (Also applicable for the CorrectSit™ CM.)

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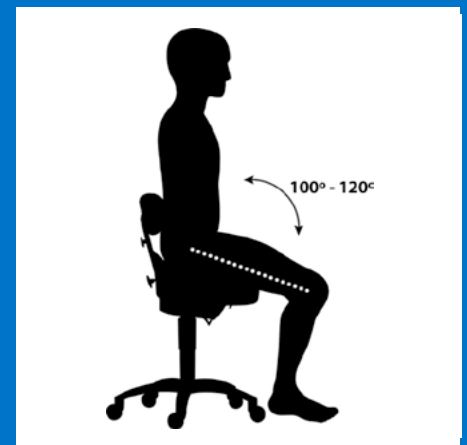EN If you've chosen the armrests, then the stool is already prepared for them.

BQE 01 / BQE CM

Place the armrest in the coupling under the stool, adjust the distance and fix it with knob (1). Determine the height of the armrests by pulling up and releasing knob (2).

BQE 02

Place the angle piece in the coupling under the stool, adjust the distance and fix it with knob (1). Place the armrest in the angle piece, turn the knob (2) loose, determine the height of the armrest and fix it with knob (2). The right angle position can be adjusted by repositioning bolt (3).

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EN The vinyl upholstery can be cleaned with a soft brush and soft soap. Do not use cleaners that contain more than 3% alcohol or oil products, it can effect the flexibility of the upholstery. It's best to clean the wheels regularly with a brush and soft soap.

Dynamic frame

In order to keep the hinges flexible we advice to smear the hinges once in 6 months with a synthetic oil. For example WD40 or HHS200. Please contact us for general servicing.

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